

ImmBioMed GmbH & Co. KG

Bergstrasse 85, DE-64319 Pfungstadt Phone: +49 6157 9799 685, Fax: +49 6157 86443

Antithrombin III Deficient Plasma

Product No. ADG203

Lot xxxxxx

Description

Antithrombin III Deficient Plasma is manufactured from pooled normal human plasma depleted of Antithrombin III using antibodies directed to Antithrombin III. Only the highest quality citrated plasma is used as starting material.

Specification

Activity assay	Lot xxxxxx
PT	xx.x sec
APTT	xx.x sec
Fibrinogen (Clauss determination)	> 200 mg/dL
Factor II	50 – 100 %
Factor V	50 – 100 %
Factor VII	50 – 100 %
Factor VIII	50 – 100 %
Factor IX	50 – 100 %
Factor X	50 – 100 %
Factor X	50 – 100 %
Factor X	50 – 100 %
Factor X	<0.1 U/mL *

^{*}All clotting assays were performed on the IL-ACL TOP using IL reagents. ATIII test kit (#0020030100)

Application

For research assays of fibrinogen antigen and functionality, as a negative control and for dilution of standards and test samples.

Warning

The plasma is prepared from normal citrated human plasma. The starting material has been tested and found negative for HIV-1/-2 antigen(s), STS, antibodies to HIV and HCV, and non-reactive for HbsAg, HCR RNA and HIV-1 RNA.

As no known test method can provide complete assurance that products derived from human blood will not transmit HBsAg, HCV, HIV-1, HIV-2 or other blood-borne pathogens, this reagent should be handled as recommended for any potentially infectious human specimen.

Presentation

Screw-capped vial containing 1 mL frozen plasma.

Preparation and Stability

Thaw 1 ml vial in 37 °C water bath for 5 minutes. Product is stable until date stated on vial label when stored at -30 °C. Once thawed, plasma is stable for 4 hours at 2-8 °C in original vial.

Storage

Plasma is shipped frozen and should be stored at $-30 \, ^{\circ}\text{C}$.

Distributed by:

LOXO & SOLEKULARBIOLOGIE

BIOCHEMIE • PRODUKTE UND SYSTEME

Gerhart-Hauptmann-Str. 48 69221 Dossenheim

Tel +49 6221 868023 Fax +49 6221 8680255 www.loxo.de - info@loxo.de

Hinweis/Note:

Der Packungsbeileger dient nur als erste Information. Der relevante Packungsbeileger liegt der Ware bei. The datasheet is for information purposes only. The current datasheet will be enclosed with product shipment.