

Factor XII Deficient Plasma

Product No. ADG212

Lot XXXXXX

Description

Factor XII Deficient Plasma is manufactured from pooled normal human plasma depleted of Factor XII using antibodies directed to Factor XII. Only the highest quality citrated plasma is used as starting material.

Specification

The plasma is assayed to ensure the proper activity of the remaining factors (>50% activity), and is tested for fibrinogen, PT, aPTT. *Please note: Results may vary depending on materials and methods used.

| Activity assay | Lot XXXXXX * |
|----------------|--------------|
| PT | XXX sec |
| aPTT | XXX sec |
| Fibrinogen | XXX mg/dL |
| Factor II | XXX % |
| Factor V | XXX % |
| Factor VII | XXX % |
| Factor VIII | XXX % |
| Factor IX | XXX % |
| Factor X | XXX % |
| Factor XI | XXX % |
| Factor XII | < 1 % |

Application

For research assays of Factor XII antigen and functionality as a negative control and for dilution of standards and test samples.

Warning

The plasma is prepared from screened normal citrated human plasma that have been tested and found negative for HIV and Hepatitis by FDA-approved methods.

As no known test method can provide complete assurance that products derived from human blood will not transmit HBsAg, HCV, HIV-1, HIV-2 or other blood-borne pathogens, this reagent should be handled as recommended for any potentially infectious human specimen.

Presentation and Storage

Set of 10 clear, screw-capped vials each containing 1 mL of frozen plasma.

Preparation and Stability

Thaw 1 ml vials in 37 °C water bath for 5 minutes. Product is stable until date stated on vial label when stored at -20 °C. Once thawed, plasma is stable for 4 hours at 2-8 °C in original vial.

Storage

Plasma is shipped frozen and should be stored at -20 °C or colder.

Distributed by:

LOXO GMBH

IMMUNOLOGIE • MOLEKULARBIOLOGIE
 BIOCHEMIE • PRODUKTE UND SYSTEME

Gerhart-Hauptmann-Str. 48
 69221 Dossenheim

Tel +49 6221 868023
 Fax +49 6221 8680255

www.loxo.de - info@loxox.de

Hinweis/Note:

Der Packungsbeileger dient nur als erste Information. Der relevante Packungsbeileger liegt der Ware bei.
 The datasheet is for information purposes only. The current datasheet will be enclosed with product shipment.

For research use only!

ADG212©08072019_Lot